ENGLISH

Read the instructions carefully before use. The novoglan® medical device is indicated for the treatment of adult phimosis and the preferential loosening of the foreskin.

Initial Preparation

- Remove the following items from the kit: 1x Novoglan Guide Insertion Rod (with White spacer), 2x O-rings, 1x Stop Cock (tap), 1x Device body (the device body is pre-assembled with one Silicone NoVo Inflator balloon), 1x Inflation Pump, 1x cotton bud and 1x Alcohol Swab.
- The spare 3x Silicone NoVo Inflator Balloons and 2x O-rings can be used as replacements, should the original pre-assembled Balloon burst or tear.
- **Note:** The kit also includes 8x latex rubber balloons, which some men may find more effective if the Phimotic ring is located further down towards the penile shaft. The flexible latex balloon can be inserted further inside the foreskin space and can provide a wider area of stretching. These latex rubber balloons are also included in the Kit for use in the preferential loosening of the foreskin. The O-rings are compatible with both the silicone NoVo Inflators and latex balloons.

Cleaning

- Gently clean the foreskin and glans (head of penis) with warm water.
- Use a cotton bud to gently remove any smegma (a white/yellowish debris) that might be present under the foreskin. Dispose of the cotton bud safely in the trash after use (each bud is for single use only).
- If the glans is sensitive due to prolonged lack of exposure, gently touch the glans with the tip of a cotton bud for a few seconds, repeating this process several times a day for 3 days before using the device. This will help desensitize the glans and make the treatment experience more convenient.

Device Preparation

- Connect the blue Stopcock to the white flexible tube of the Novoglan Inflation Pump, by twisting it clockwise. Then put it aside.
- Use an alcohol swab provided, to clean and sanitize the entire surface of the Silicone NoVo Inflator Medical Grade Balloon on the pre-assembled Device body. Ensure the swab contacts the entire balloon. Dispose of the swab in the trash after use. **Note: The Silicone NoVo Inflator balloons are reusable but must be cleaned before each use with an alcohol swab provided.**
- Gently insert the white Novoglan Guide Rod through the end of the pre-assembled Device body, until it is fully engaged. Be careful not to force the rod in or damage the balloon wall with the end of the rod.

Phimosis Treatment

- The minimum treatment time per session is 15 minutes with a maximum duration not to exceed 30 minutes.
- Daily treatment is recommended. Your doctor may recommend treating twice daily with at least 8 hours between each treatment session.

Device Insertion

- Apply a very small amount of water-based personal lubricant to the top of the NoVo Inflator Balloon to facilitate the introduction of the balloon under the foreskin.
(<https://shop.novoglan.com/products/novoglan-personal-lubricant>)
- Holding the base of the engaged Guide Rod, carefully position the tip of the prepared NoVo Inflator Balloon into the space under the foreskin and between the glans, directly above the urethra (the hole from where urine comes out).

Scan here to watch the video


Note: i) Do NOT insert the NoVo Inflator Balloon into the urethra. ii) The Device must remain in the one selected position for each specific treatment session. iii) Some men may experience some discomfort in the initial stages of treatment - this may be due to the glans being very sensitive if not regularly exposed. This discomfort should not persist with repeated use of the Device.

- Insert the NoVo Inflator Balloon just far enough to be comfortable and held in- place.
- Those men with only a 'pinhole' opening (refer to Grade 1 Phimosis in the Phimosis Grade Chart below), may need to take extra care to gently wriggle the tip of the NoVo Inflator Balloon with the Guide Rod inserted, so it just sits inside the 'pinhole' space. **Do NOT force it in deeply.** The Novoglan Device is designed to treat all grades of Phimosis, including 'Pin-hole'.

Device Activation

- Once the NoVo Inflator Balloon is correctly positioned between the foreskin and glans, remove the Guide Rod and store in the tray provided, for next use.
- Attach the Novoglan Inflation pump by inserting the tip of the Stopcock, into the rear of the Device, taking care not to accidentally dislodge the balloon. Ensure the tap is open. **Note: You may need to vigorously twist the tap back & forth to loosen before first use.**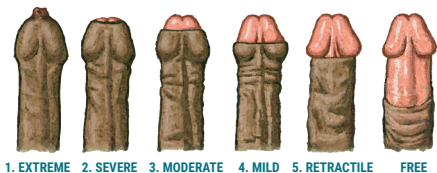

- Lightly squeeze the Pump bulb to inflate the NoVo Inflator Balloon just enough to gently stretch the foreskin. Start with a small amount of inflation, then adjust the pressure to suit your own comfort level as your treatment progresses.
- Twist the Stopcock tap to lock off the inflated Balloon.
- Detach the pump from the Stopcock by twisting the white flexible tube anti- clockwise. The Device should now remain inflated between the glans and the foreskin.
- Keep the Device in the same position for a maximum duration of 30 minutes per treatment session (*Your doctor may suggest an alternative regimen of twice daily for only 15 minutes per session*). Although it is designed to slowly leak and prevent excessive pressure build-up, **never sleep with the No-vo-glan device inserted.**

Note: Rarely an inserted inflated balloon may burst under the foreskin. The loud noise and rush of air across the sensitive glans, may cause some fleeting discomfort. Immediately remove and discard this balloon, and attach a new one (refer to Replacing Silicone NoVo-Inflator Balloon steps outlined below)


INSTRUCTIONS FOR USE

novoglan®

Device Removal

- To remove the Device, simply turn the Stopcock tap to the open position and deflate the Balloon. Then gently pull it out from under the foreskin. Once removed, store all components safely in the storage tray, out of reach of children.

Repeat these steps each day for a period of 6 to 8 weeks. **Note: If you have any questions or concerns about your treatment, please contact your Healthcare professional.**

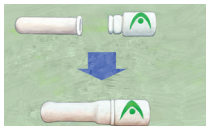Replacing Silicone NoVo-Inflator Balloon

- Immediately discard any damaged or torn NoVo Inflators by first removing the 2x O-rings and peeling the Balloon off the Device body.
- Take a new clean Silicone NoVo Inflator Balloon then attach it over the indented end of the Device body, being careful not to tear or damage the Balloon wall.
- Slide both O-rings down over the Balloon until secured in place at the base of the Device body.
- Attach the pump and test for leaks before commencing treatment. Adjust the O-rings for a tighter seal if necessary.

PRODUCT WARNINGS | READ THIS PRIOR TO FIRST USE:

*** WARNING – NATURAL RUBBER LATEX INFLATION BALLOONS ARE IN THIS KIT

DO NOT use the NOVOGLAN Device if you have **allergies to latex**, rubber or any other synthetic materials (e.g. silicone). **DO NOT** use the NOVOGLAN Device for any other purpose other than as indicated for use

DO NOT forcibly pull back the foreskin at any time. **If you pull back a tight foreskin before it is ready to be retracted, it can get stuck in the retracted position.** This can lead to a condition known as **Para-phimosis** - where the foreskin is permanently retracted, and this constricts the blood flow and can lead to severe complications. If you think this has happened to you, **seek URGENT medical attention.**

DO NOT use or continue to use the NOVOGLAN Device if you experience any of the following: pain, redness, infection, herpes, warts, bleeding or discharges of any kind. Please consult your medical practitioner immediately if you experience any of these.

The body of the NOVOGLAN Device is manufactured from moulded high shore (75) silicon. The NOVOGLAN Silicone NoVo Inflators are natural Silicone Rubber. Medical grade Latex NoVo Inflator Balloons are included. The stopcock is made from medical grade plastic. Kits will have a minimum of 3 Silicone NoVo Inflator Balloons. Some retail or combination packs will have more than 3 extra balloons.

NOVOGLAN Device Manufactured in Australia NOVOGLAN Kit Packaged and Distributed by: Platigo Solutions Pty Ltd Suite 802, 50 Clarence Street Sydney, NSW 2001, Australia. This document is current at 11-JUL-2023. Go to website for updates. www.novoglan.com

NOVOGLAN is a Trademark of IPH001 Pty. Ltd. and used by Platigo Solutions Pty. Ltd. under license. Medical Device Regulations: US FDA# 00860009967205 / AUS ARTG# 168962 / CE Marketing Approval / UK MHRA# CA015054 ^ Updated Clinical Trial Data reported April 2022.

HOW TO CONTACT Novoglan Customer Service Team.

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